

KNEAL KNEE CRUISER BASKET INSTALL

CONTENTS	1	Kneal Knee Cruiser Basket
	1	5mm wrench
	1	lock nut
	1	handle pad (optional)
	1	safety strap (only used on extended handle)

SETUP

1. Loosen brake levers with 4mm hex wrench. **(A)**
2. Once loose, gently push down on levers. They should be in the approximate position shown in image. **(B)**
3. Attach the bracket using the 5mm hex wrench. It should be tilting ever so slightly backward. See Image. **(C)**
4. Tighten until bracket will not move on handlebars. **(D)**
5. Attach basket. Be sure that levers clear the back of the basket.
6. Remove basket and retighten levers. **(E)**

